

# Checklist for reconditioning of electric wheelchairs for indoor use

Electric wheelchair ..... Series no .....

Users .....

Recommended service interval 1 times / year or more often if the user is heavier or spastic, as this may cause more stress on the wheelchair.

**Electronics and electrical systems**

(Testing of control systems, cleaning, adjusting if necessary. Checking actuators, fixings, enclosures, cabling, etc.



**Wheels and suspension**

(Check wear and tear, cleaning, adjusting, lubricating, changing wheels, wheel fork, wheel bolt, etc.)



**Lifting pillar Battery**

(Fastening, drying out, otherwise maintenance-free) (Check status, condition, contacts, any load test)



**Knobs**

**Seat and back mechanism**

(Lubrication and cleaning) (checking attachments, wear, gaps, cracks, etc. lubrication)



**Armrest Upholstery**

(Cleaning, possible replacement of armrest plates) (Cleaning or washing 60 °, change if necessary.)



**Leg support**

(Cleaning, checking wear, condition)



**Brake release**

**Possible extra equipment**

(Functional control, adjustment and lubrication of wires)

(Cleaning, Lubrication, Functional Testing)



**Test run & function test**

(Steering, braking, charging and all controls)



Checked on ..... / ..... / .....

Checked by .....

Signature

**Eurovenma**

**Creates mobility and ergonomics.**