

Reconditioning checklist, rollator

Volaris rollators are intended to be reused throughout their expected service life (7 years).

Cleaning


Clean in a washing cabinet at 60° for 45min with the brakes disengaged. The rollator must be allowed to cool to room temperature before being adjusted and used. Manual cleaning using a high pressure washer, water hose, cloths impregnated with a mild detergent with a pH value of 7-12, e.g. soap solution, or appropriate non-abrasive cleaning cloths. Surface disinfectant of an approved grade may be used. Let the disinfectant sit for 1– 5 minutes and leave to air dry.

See the reconditioning instructions for a more detailed description of the points below.

1. Wheel

Approved

Comments

1.1	Tread		
1.2	Rim		
1.3	Wheel hub		
1.4	Wheel bearings		
1.5	Wheel fork bearings		
1.6	Wheel forks, cracks, damage		
	Wheel bolts must be replaced with new ones if they come loose		

2. Brakes

2.1	Brake shoe, condition		
2.2	Braking effect, adjustment		
2.3	Return spring		
2.4	Cables		
2.5	Functional checks		

3. Push handle**Approved****Comments**

3.1	Handle grip		
3.2	Height adjustment		
3.3	Aluminium profiles		

4. Folding mechanism

4.1	Aluminium cross		
4.2	Fold lock		
4.3	Functional checks		

5. Seat

5.1	Height adjustment		
5.2	Condition		

6. Chassis

6.1	Aluminium profiles, chassis		
-----	-----------------------------	--	--

7. Extra check, Volaris RA

7.1	Armrest cushions		
7.2	Push handle		
7.3	Lock wheels/lock screws.		

Rollator model: _____

Serial number: _____

Reconditioning carried out by:

Name: _____

Date: _____

Signature: _____