

A family of four is walking along a cobblestone path next to a rustic stone wall. A man in a light blue polo shirt and black shorts walks on the left, carrying a grey bag. A woman in a blue sleeveless top walks behind a black stroller. A young girl in a red t-shirt and white skirt walks on the right, holding the stroller's handle. A baby wearing glasses and a white shirt sits in the stroller. The scene is bright and sunny.

Viola & Grillo Postural System




Postural Seating System for children from 75 to 150 cm tall.

Available in 3 different sizes of the seats (mini, small, medium) that you can fix on two sizes of bases (for indoor or outdoor uses).

Have a look of the story of our young testimonial in the following video

3 sizes, 2 frames for outdoor use
3 colors



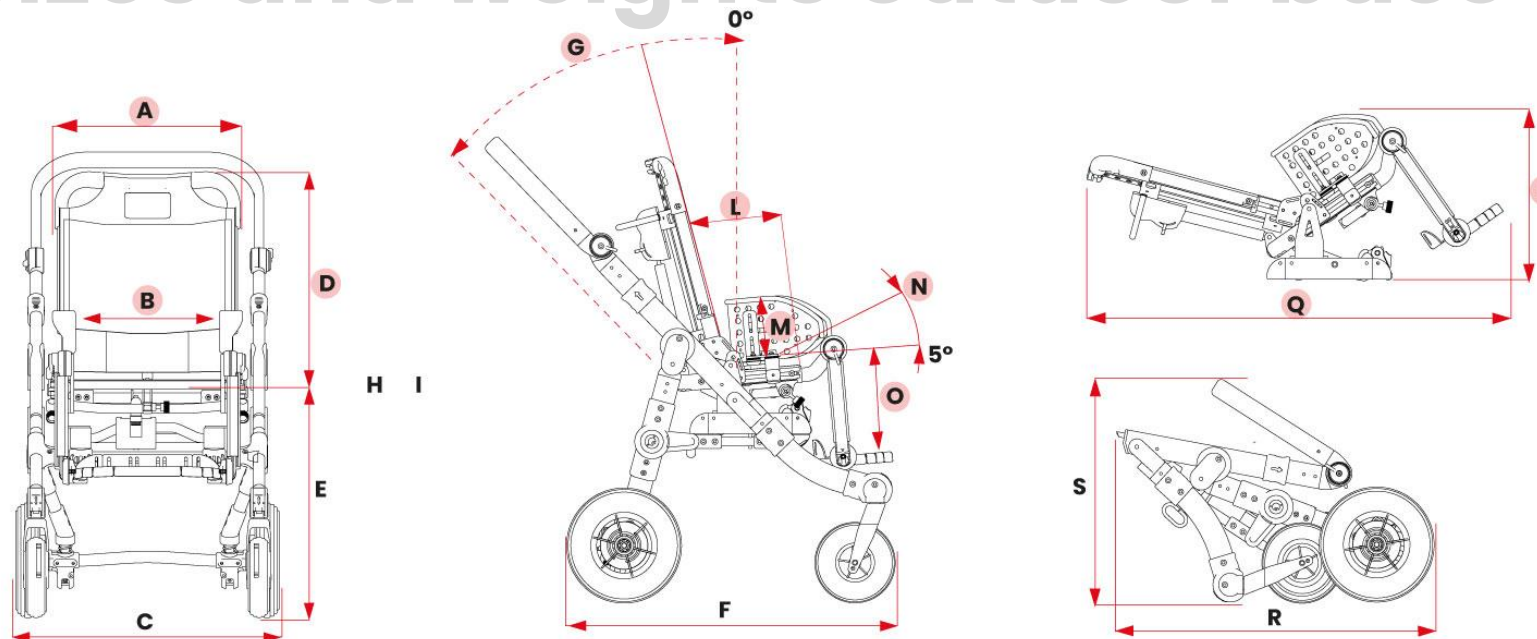

SIZES	[A] backrest width (cm)	[B] Seat width (cm)	[C] width dimensions (cm)	[D] min. max height backrest (cm)	[E] height seat from ground (cm)	[F] depth dimensions (cm)	[G] backrest reclination (degrees)	[L] seat depth (cm)	[M] armrest height (cm)	[N] seat tilt (degrees)	[O] distance seat- footrest (cm)	P (cm)	Q (cm)	R (cm)	S (cm)	[V] User height (cm)	Max load (kg)	Weig ht of frame [kg]	Weight of seat [kg]	Total weight [kg]
MINI	31	16-26	57	42-52	51	72	45° (da -5° a 40°)	16-26	11,5-14	30° (da 5° a 35°)	15-25	31	65	62	49	75-105	35	9,8	8,8	18,6
SMALL	37	20-30	57	48-62	52	72	45° (da -5° a 40°)	20-30	13-16,5	30° (da 5° a 35°)	21 - 31	33	74	64	49	90-120	35	9,8	9,9	19,7
MEDIU M	43	25-35	63	60-75	55	81	45° (da -5° a 40°)	28-40	16-20	25° (da 5° a 30°)	26-35 (fronte mamma) 26-38 (fronte strada)	34	88	65	58	115- 150	45	10,8	12	22,8

Supportive, long-lasting and breathable cover of the seat

That grants

› **Hygiene:** thanks to the excellent breathability, it reduces sweating and bacteria formation.

› **Washing:** despite the supportive cover is not removable (the upholstery is), the fabric composition allows easy sanitization and washing.

› **Quick drying.**

› **Resistance:** High resistance to abrasion. The high resistance texture maintains its performance even at high loads (ISO 12947 test).

› **Fire retardant:** according to EN 1021-1:2014 and EN 1021-2:2014.





High performance upholstery, removable and easy to wash!

Optimal "self supporting effect": the padding structure, highly innovative, allows optimal weight distribution and excellent comfort.

Light and breathable: ensures extremely high air circulation, reducing sweating and enabling the thermoregulation.

Hygienic: easy to wash with quick drying times.

Abrasion resistant: The high-strength texture maintains its performance even at high loads (ISO 12947 test).

Fire retardant: according to EN 1021-1:2014 and EN 1021-2:2014 standards.

OEKO-TEX certificate: for all fabric that are in direct contact with the skin, to guarantee quality and safety for health.

The new upholstery has a compressive strength (DIN EN ISO 3386) 70% higher than foam.

Backrest shape

Shaped and supportive backrest

to accommodate the
trunk giving appropriate
support thanks to the
resistant cover and the
two inner **aluminum
lateral bars**

granting maximum
breathability and
comfort at the same
time.




Backrest back view



Backrest top view

Backrest/seat shape

the backrest and the seat
are designed to attach
shaped cushions/foams

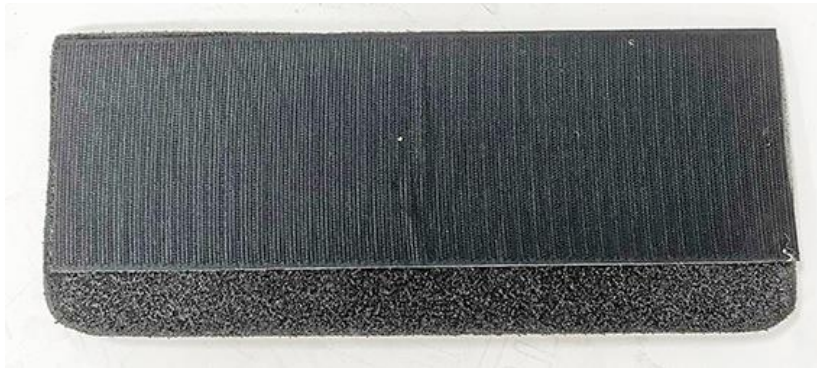


shaped cushions



GRILLO
POSTURAL SYSTEM

Posture control: reversible seat

ORMESA®
• MADE IN ITALY SINCE 1980 •

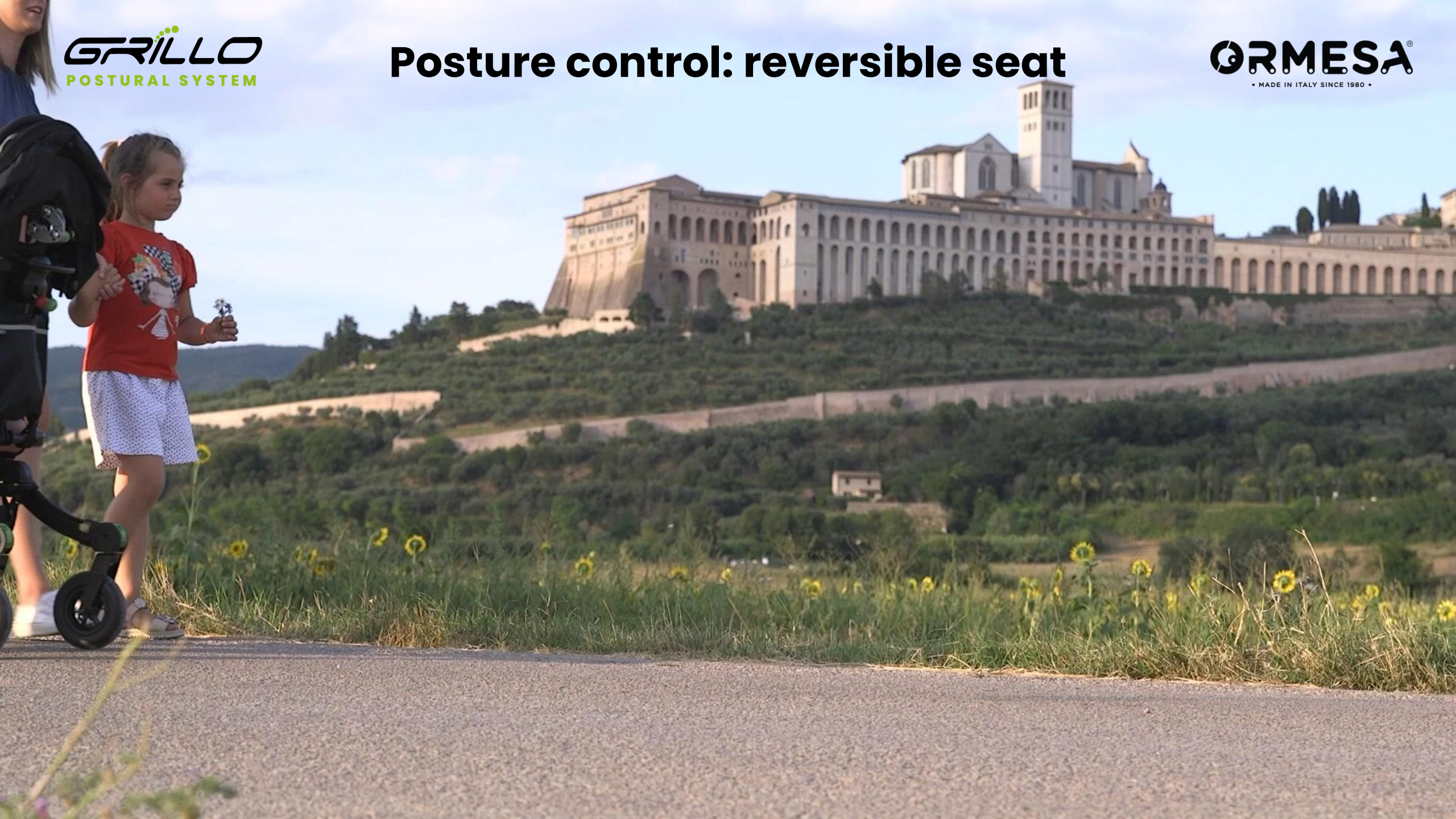



The mechanical joints are close to the anatomical ones to ensure comfort during adjustments.

easy and safe to manage, for each size



Posture control: reclining

operated by gas spring: stepless and comfortable



Posture control: tilt in space

operated by gas spring: stepless and comfortable



Very safe and easy to handle locking system



Green signal – Locked



Red signal – Unlocked

attach and detach the seat



Stroller closed with seat attached



Stroller closed with seat removed



Closing the frame





Parking brakes



Direction locks



Trunk supports adj
in height, rotation
and width



Wrappable trunk
supports adj in
height and width



Two points pelvic
belt



Four points pelvic
belt




Vest harness



Adj. in height, inclination, backward-forward, with occipital adjusted supports. It is also horizontally adjustable




Adj. in height, inclination, backward-forward, width parietal adjusted supports. It is also horizontally adjustable

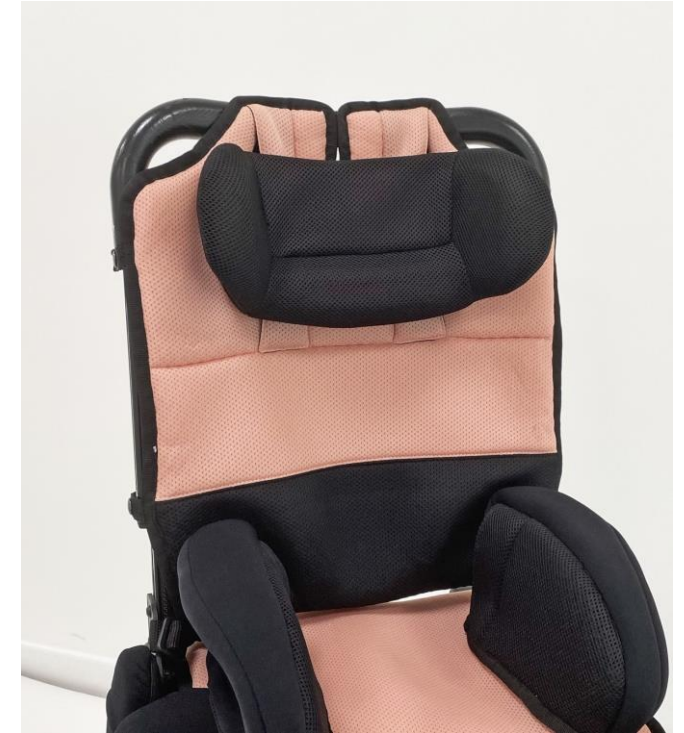

Shaped and padded headrest, adj. in height



Abductors:
adj in width and
the simply narrow one

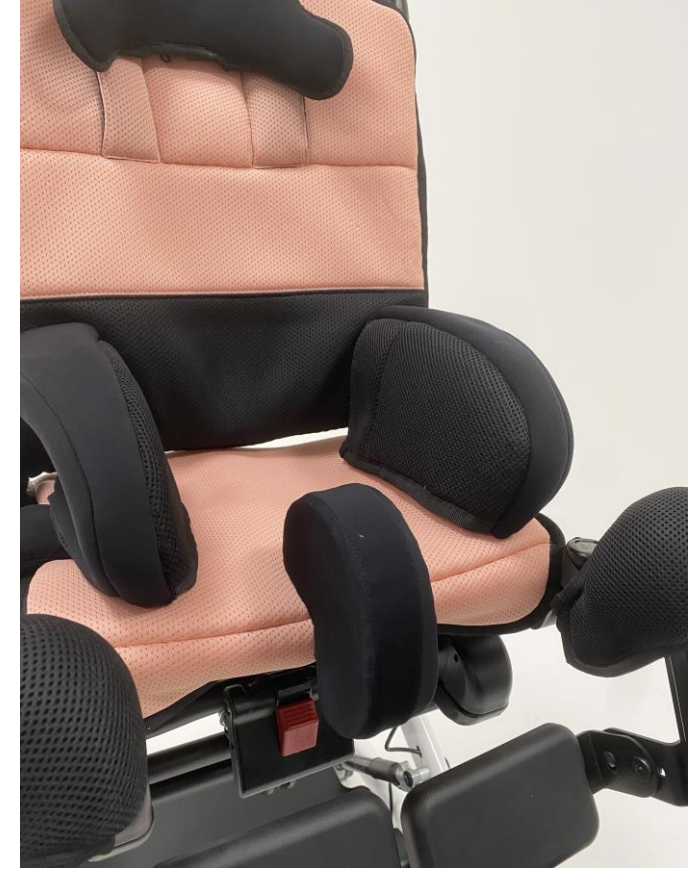


Two different kind of legrests: simple and separated one



Single footrest
Adj. in height
and tilt





Separated Legrests adj in height,
tilt and abduction independently

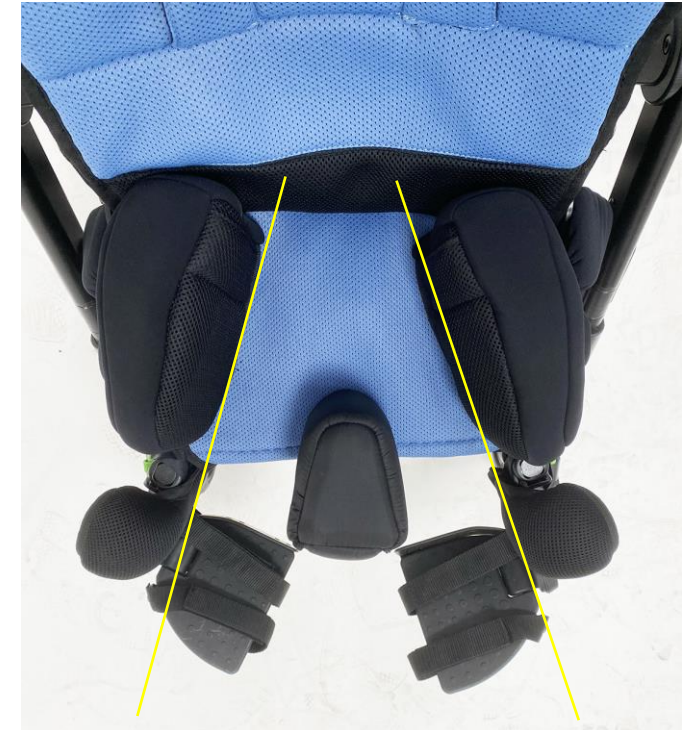


Adductive and Abductive positions

Side pelvic supports adj in Depth, Height, Width and rotation (previous slide)

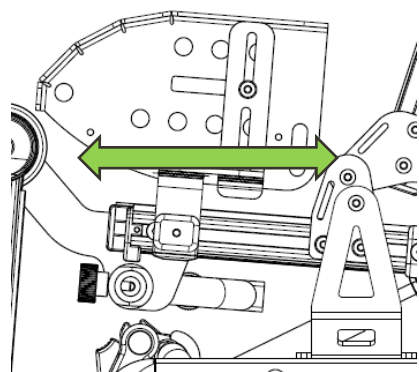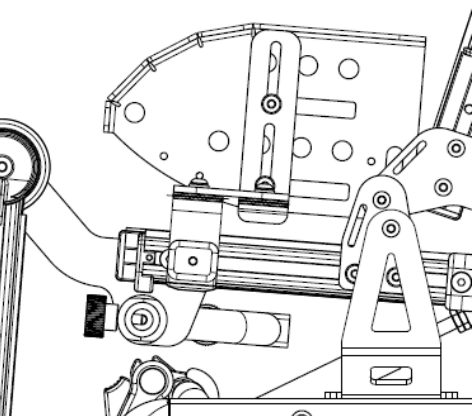

Depth to follow
the depth of the
seat

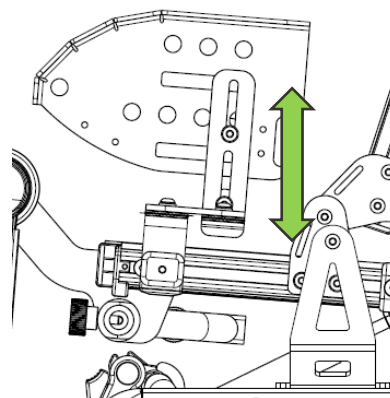

Height





Table adj in depth, height, tilt with the armrests; resistant, light and easy to sanitize. Indicated for indoor use



Transparent table for indoor and outdoor use; same adjustments



Front handle adj in tilt and height



Oxygen bottle holder



Basket and vent holder



Sibling rear footboard



Hand brake



Thermal cover



Rain cover



Canopy

Frame: adjustable handlebar

