



Important information for users of:

Miniflex SitRite, Flexbase SitRite, Flexbase Forma with seat cross as shown in the picture.

During the period **01.09.2022 to 15.10.2024**, seat crosses may have been supplied with a temporary weld, potentially causing the front attachment lip on the seat lock and tilt mechanism to come loose from its mounting. This may result in the user falling out of the seat and therefore constitutes a safety hazard. **There is also a risk of this, for seat cross with a defective torque package. All products that may be affected by both deviations are listed below.**

Please notify all affected individuals in your organization.

Eurovema Mobility AB is able to trace relevant models and will be contacting affected customers during November and December to discuss how the problem will be addressed. **See also supplementary serial numbers below in bold.**

Remedial measures will be carried out either by Eurovema technicians or third party engineers contracted by Eurovema, who will remove the seat cross and fit a correctly designed replacement. The plan is to carry out works on affected chairs during 2025.

If you have any questions, please contact Eurovema Customer Services, either via e-mail at info@eurovema.se or telephone +46 (0)371 39 01 00

Quality Manager: peter.kjell@eurovema.se

Important Safety Notice to the Market

Safety Notice: FSN 104122
2025-03-12

Affected serial numbers:

1040425848	1040430127	1040430714	1040431199	1040431612
1040429309	1040430137	1040430773	1040431204	1040431613
1040429310	1040430180	1040430781	1040431206	1040431630
1040429346	1040430197	1040430786	1040431253	1040431671
1040429349	1040430201	1040430791	1040431254	1040431672
1040429352	1040430213	1040430847	1040431261	1040431780
1040429466	1040430215	1040430849	1040431262	1040431781
1040429471	1040430220	1040430853	1040431265	1040431782
1040429474	1040430226	1040430854	1040431267	1040431801
1040429484	1040430227	1040430856	1040431268	1040431830
1040429494	1040430288	1040430860	1040431321	1040431837
1040429495	1040430289	1040430861	1040431322	1040431838
1040429497	1040430293	1040430869	1040431324	1040431839
1040429507	1040430294	1040430871	1040431325	1040431840
1040429509	1040430295	1040430872	1040431326	1040431841
1040429510	1040430298	1040430875	1040431327	1040431866
1040429546	1040430301	1040430896	1040431328	1040431867
1040429549	1040430303	1040430897	1040431333	1040431868
1040429550	1040430319	1040430898	1040431356	1040431871
1040429636	1040430488	1040430905	1040431359	1040431872
1040429651	1040430492	1040430908	1040431403	1040431874
1040429652	1040430500	1040430909	1040431408	1040431875
1040429663	1040430501	1040430910	1040431409	1040431879
1040429664	1040430502	1040430911	1040431413	1040431880
1040429666	1040430503	1040430930	1040431428	1040431881
1040429667	1040430504	1040430932	1040431449	1040431888
1040429702	1040430535	1040430934	1040431459	1040431889
1040429703	1040430537	1040430940	1040431460	1040431891
1040429704	1040430550	1040430992	1040431481	1040431892
1040429745	1040430604	1040430994	1040431490	1040432041
1040429937	1040430605	1040431101	1040431491	1040432042
1040429940	1040430609	1040431102	1040431492	1040432043
1040429942	1040430611	1040431103	1040431493	1040432045
1040429946	1040430620	1040431104	1040431494	1040432047
1040429957	1040430622	1040431144	1040431495	1040432111
1040429958	1040430671	1040431148	1040431496	1040432114
1040430050	1040430676	1040431164	1040431571	1040432115
1040430051	1040430677	1040431187	1040431572	1040432116
1040430056	1040430689	1040431189	1040431600	1040432119
1040430060	1040430691	1040431190	1040431608	1040432121
1040430063	1040430692	1040431196	1040431609	1040432133
1040430065	1040430711	1040431197	1040431610	1040432135
1040430066	1040430712	1040431198	1040431611	1040432143

Important Safety Notice to the Market

Safety Notice: FSN 104122
2025-03-12

1040432150	1040433104	1040433483	1040434054	1040434490
1040432179	1040433106	1040433570	1040434059	1040434491
1040432191	1040433128	1040433577	1040434076	1040434494
1040432194	1040433130	1040433578	1040434083	1040434495
1040432223	1040433134	1040433587	1040434084	1040434496
1040432224	1040433135	1040433588	1040434085	1040434497
1040432227	1040433136	1040433589	1040434088	1040434499
1040432234	1040433137	1040433629	1040434089	1040434512
1040432235	1040433138	1040433654	1040434103	1040434515
1040432236	1040433139	1040433655	1040434104	1040434516
1040432237	1040433140	1040433656	1040434107	1040434519
1040432238	1040433141	1040433657	1040434143	1040434520
1040432241	1040433142	1040433662	1040434150	1040434521
1040432242	1040433153	1040433679	1040434159	1040434522
1040432243	1040433156	1040433692	1040434160	1040434523
1040432251	1040433215	1040433697	1040434163	1040434524
1040432252	1040433221	1040433745	1040434174	1040434525
1040432254	1040433228	1040433751	1040434183	1040434526
1040432271	1040433266	1040433757	1040434189	1040434566
1040432272	1040433276	1040433758	1040434268	1040434567
1040432284	1040433286	1040433761	1040434312	1040434571
1040432295	1040433287	1040433765	1040434313	1040434572
1040432303	1040433290	1040433789	1040434314	1040434585
1040432304	1040433298	1040433796	1040434327	1040434586
1040432306	1040433350	1040433801	1040434348	1040434587
1040432396	1040433351	1040433804	1040434372	1040434588
1040432397	1040433355	1040433807	1040434388	1040434589
1040432398	1040433416	1040433818	1040434392	1040434590
1040432403	1040433417	1040433820	1040434393	1040434591
1040432405	1040433418	1040433832	1040434420	1040434594
1040432406	1040433419	1040433834	1040434422	1040434625
1040432445	1040433420	1040433870	1040434424	1040434632
1040432447	1040433429	1040433900	1040434429	1040434944
1040432515	1040433442	1040433902	1040434430	1040434946
1040433001	1040433444	1040433903	1040434455	1040434957
1040433002	1040433445	1040433915	1040434475	1040434961
1040433003	1040433446	1040433922	1040434479	1040434962
1040433042	1040433471	1040433934	1040434481	1040434963
1040433057	1040433473	1040433941	1040434482	1040434978
1040433059	1040433474	1040433950	1040434483	1040434997
1040433071	1040433475	1040433952	1040434484	1040435032
1040433072	1040433476	1040433957	1040434485	1040435033
1040433087	1040433477	1040433958	1040434486	1040435064
1040433092	1040433478	1040434023	1040434487	1040435067
1040433102	1040433479	1040434050	1040434488	1040435068
1040433103	1040433480	1040434051	1040434489	1040435121

Important Safety Notice to the Market

Safety Notice: FSN 104122
2025-03-12

1040435123	1040435676	1040436061	1040436829	1040437318
1040435126	1040435677	1040436066	1040436830	1040437319
1040435127	1040435678	1040436078	1040436831	1040437321
1040435132	1040435679	1040436107	1040436832	1040437322
1040435134	1040435680	1040436113	1040436857	1040437323
1040435146	1040435681	1040436115	1040436858	1040437324
1040435195	1040435722	1040436116	1040436859	1040437366
1040435240	1040435773	1040436121	1040436860	1040437375
1040435243	1040435796	1040436135	1040436885	1040437376
1040435244	1040435798	1040436153	1040436916	1040437377
1040435245	1040435800	1040436154	1040436940	1040437378
1040435246	1040435803	1040436165	1040436943	1040437379
1040435274	1040435810	1040436174	1040436944	1040437380
1040435462	1040435813	1040436175	1040436945	1040437381
1040435534	1040435814	1040436176	1040436946	1040437396
1040435535	1040435824	1040436177	1040436947	1040437448
1040435594	1040435847	1040436178	1040437007	1040437455
1040435595	1040435874	1040436179	1040437014	1040437456
1040435596	1040435875	1040436188	1040437028	1040437457
1040435603	1040435886	1040436189	1040437029	1040437458
1040435604	1040435887	1040436190	1040437030	1040437459
1040435605	1040435889	1040436193	1040437031	1040437460
1040435608	1040435909	1040436209	1040437050	1040437461
1040435609	1040435916	1040436641	1040437102	1040437462
1040435617	1040435917	1040436664	1040437106	1040437486
1040435618	1040435943	1040436665	1040437183	1040437492
1040435630	1040435944	1040436666	1040437197	1040437499
1040435630	1040435945	1040436667	1040437209	1040437511
1040435662	1040435946	1040436700	1040437225	1040437531
1040435663	1040435947	1040436702	1040437226	1040437532
1040435664	1040435948	1040436725	1040437271	1040437544
1040435665	1040435949	1040436753	1040437273	1040437545
1040435666	1040435950	1040436756	1040437274	1040437546
1040435667	1040436000	1040436791	1040437275	1040437547
1040435668	1040436011	1040436801	1040437276	1040437548
1040435669	1040436016	1040436818	1040437277	1040437549
1040435670	1040436017	1040436824	1040437284	1040437550
1040435671	1040436031	1040436825	1040437285	1040437651
1040435672	1040436032	1040436826	1040437295	1040437652
1040435674	1040436038	1040436827	1040437314	1040437688
1040435675	1040436039	1040436828	1040437316	

Important Safety Notice to the Market

Safety Notice: FSN 104122
2025-03-12

Kompletterande serienummer:

1040390556	1040392988	1040405011	1040393135	1040405466
1040391979	1040392995	1040405020	1040393144	1040406726
1040392044	1040393027	1040405026	1040393698	1040406727
1040392048	1040392694	1040405311	1040393709	1040405300
1040397334	1040392997	1040405378	1040405376	1040406083
1040382912	1040397392	1040405379	1040405377	1040408673
1040387335	1040397553	1040405381	1040393738	1040406728
1040385006	1040397554	1040405264	1040394734	1040406731
1040387345	1040398640	1040405420	1040405380	1040406734
1040387346	1040402696	1040402442	1040405382	1040405032
1040392303	1040402746	1040404728	1040401909	1040405306
1040391362	1040402701	1040404729	1040402738	1040406733
1040393146	1040404047	1040404730	1040402886	1040408103
1040394698	1040404052	1040395624	1040405433	1040408134
1040394735	1040391604	1040402737	1040405434	1040408135
1040394741	1040392693	1040404990	1040405458	1040408606
1040386050	1040393276	1040405018	1040408138	1040408661
1040392977	1040404056	1040405723	1040408603	1040408662
1040392992	1040404352	1040405727	1040408604	1040408136
1040392994	1040404648	1040405182	1040405041	1040408594
1040391861	1040404741	1040405344	1040406724	1040408605
1040392996	1040404516	1040405346	1040406725	1040408945
1040393279	1040404735	1040405347	1040406739	1040408194
1040391577	1040404765	1040405373	1040408197	1040408692
1040392987	1040404983	1040405374	1040403962	1040408931
1040392993	1040404537	1040405375	1040404345	1040408932
1040386482	1040404992	1040392999	1040404346	1040408933
1040390027	1040397339	1040393007	1040404984	1040408946
1040390079	1040397412	1040393088	1040404647	1040408947
1040392998	1040405023	1040393089	1040405465	1040408948

Important Safety Notice to the Market

Safety Notice: FSN 104122
2025-03-12

1040408951	1040408929	1040409302	1040408675	1040413112
1040408954	1040408930	1040409303	1040412566	1040409679
1040409044	1040409549	1040409304	1040412080	1040414227
1040409050	1040409604	1040409305	1040411983	1040408881
1040408883	1040407175	1040409306	1040411984	1040408949
1040408878	1040409544	1040411119	1040411985	1040408953
1040408879	1040409539	1040411424	1040411986	1040408971
1040408925	1040409541	1040411715	1040411987	1040408972
1040408944	1040409656	1040411808	1040412567	1040409042
1040408950	1040409665	1040411977	1040412130	1040409548
1040408952	1040409666	1040411978	1040412786	1040409029
1040409020	1040409669	1040411979	1040412850	1040409058
1040409039	1040409674	1040411980	1040412853	1040416250
1040409043	1040409755	1040411981	1040412900	
1040409048	1040404343	1040411945	1040413099	
1040409049	1040407172	104011952	1040413100	
1040409232	1040409307	1040405301	1040413101	