




Monteringsanvisning Vajer sitslyft Sigma

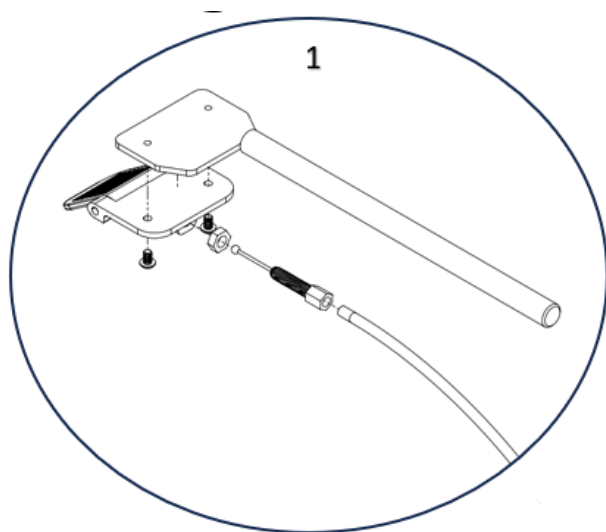

Montering av vajer sitslyft vid gaskolven

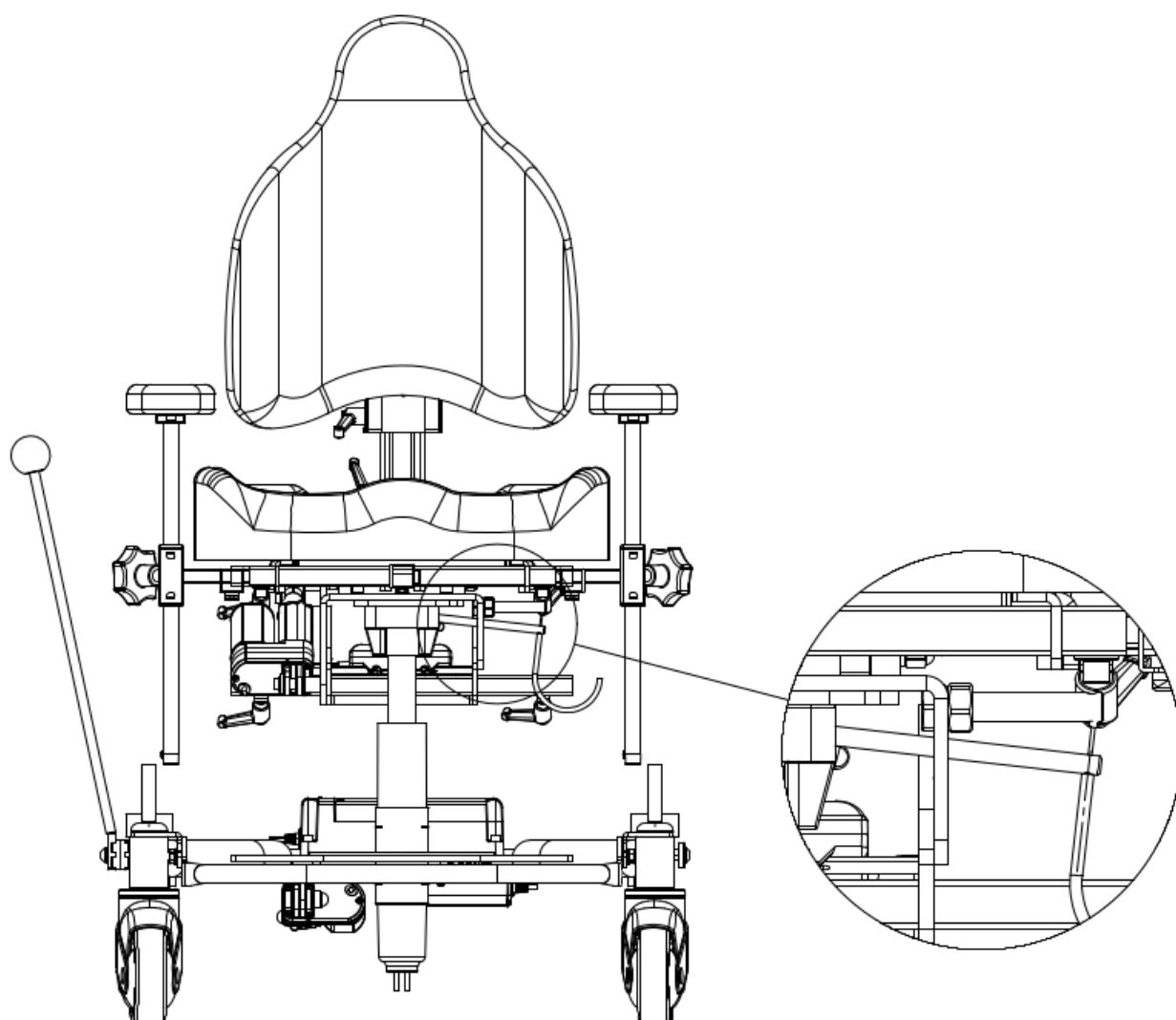

Vajern monteras i reglaget **1** genom att vajern förs genom det gängade mothållet, skruva in ställskruven så långt det är möjligt, placera vajerns kula i uttaget i reglaget.

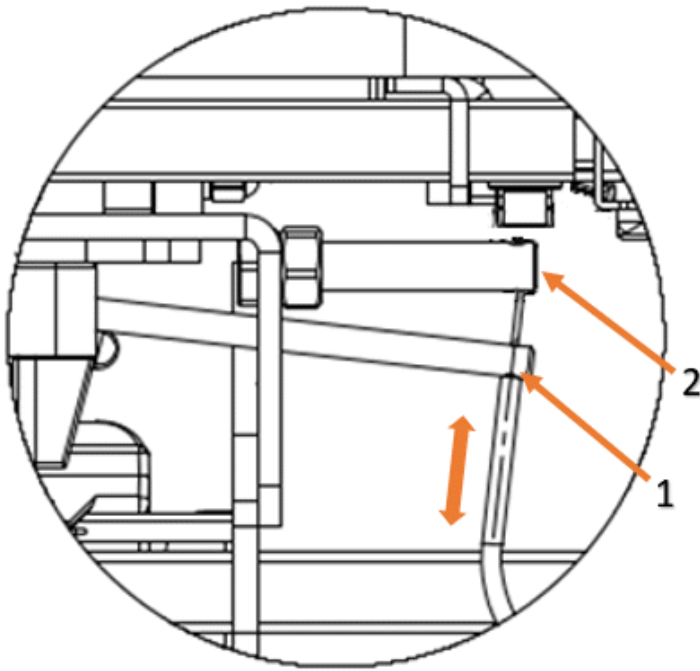

Vajer till sitslyften förs genom axeln **1** som aktiverar gaskolven och genom mothållet **2**. Lås vajern med skruvnippel, nippeln ska vara vänd med den smala sidan ner mot hålet i mothållet **2**. Spänn vajern så att reglaget under armstödet är uppspänt, lämna lite spel som slutligen justeras med spännskruven på reglaget.

Kontrollera att sitslyften fungerar korrekt vid aktivering av reglaget och att lyften ej sjunker under belastning när reglaget ej är aktiverat.

This image shows a single sheet of white paper with horizontal blue or grey ruling lines. The lines are evenly spaced and run across the width of the page. There are approximately 20 lines visible. The paper has a slight shadow on the right side, suggesting it's resting on a surface. There is no handwriting or other markings on the paper.