

Cable brake replacement instruction FSN 104121


Tools and materials:

- ***Allen key, size 5***
- ***Ring spanner, size 8***
- ***Wheel bolt (MC6S M6x20)***
- ***Wheel washer (6.4x11x1.6)***
- ***Cable ties***

Replacement of cable brakes FSN 104121 (revision 1)


Prior to installation:

1. Completely remove armrests and head rest.
2. Turn the chair upside down, e.g. on a table, to make work easier.

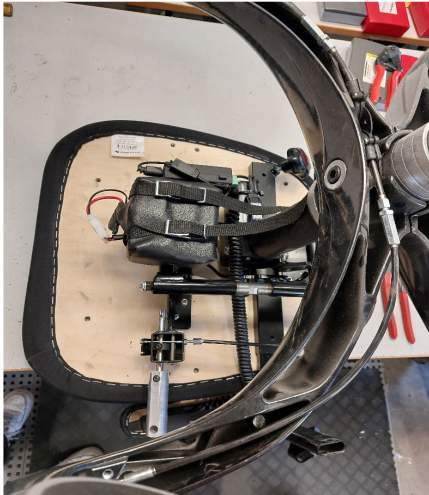


Procedure for dismantling the cable brakes:

3. Start by undoing the nut using a size 8 ring spanner.

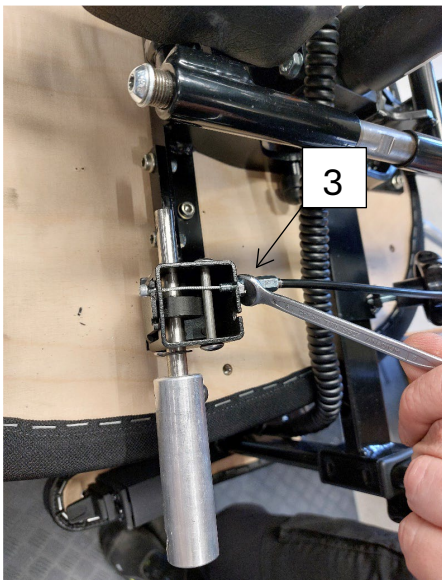


Replacement of cable brakes FSN 104121. (revision 1)

4. Turn the adjuster screw until the cable releases.
5. Remove the cable from the black mounting.

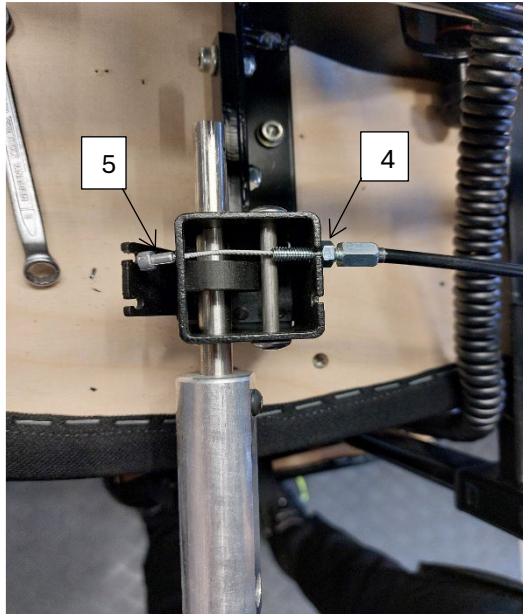


6. Unscrew the adjuster screw and thread the cable out of the brake housing.

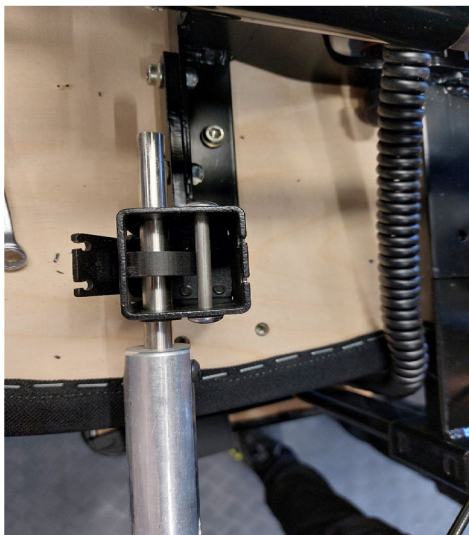


Replacement of cable brakes FSN 104121. (revision 1)

7. Cut the cable tie and pull the cable out of the chassis.

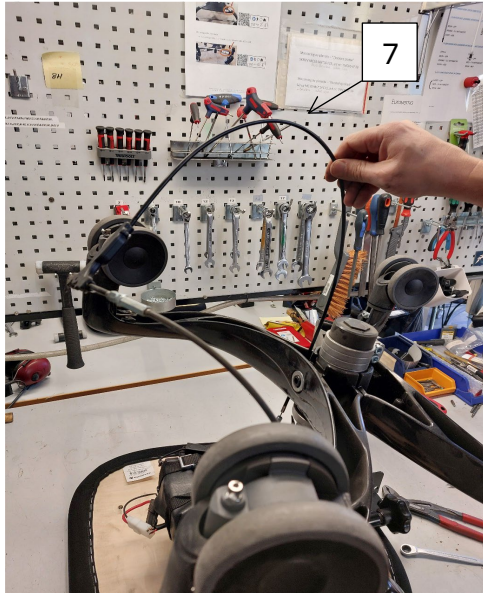


8. Undo the wheels on the side of the leg half where the cable is located.

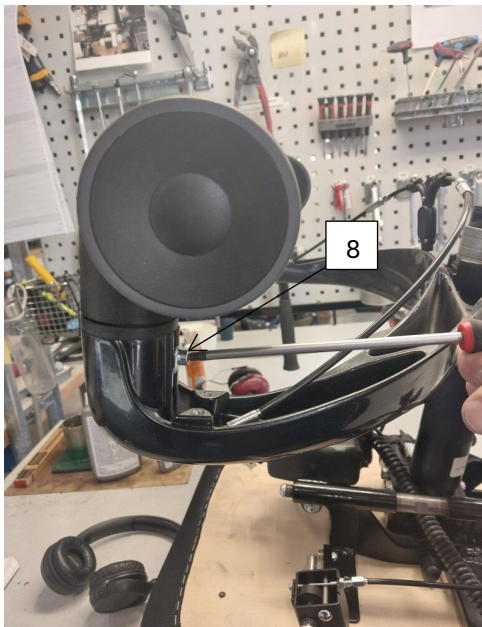

Replacement of cable brakes FSN 104121. (revision 1)

Installation of new cable brakes

9. Repeat the above procedure in reverse order
10. Secure the cable brake using a fresh cable tie. Make sure that the cable brake is positioned as high up as possible.

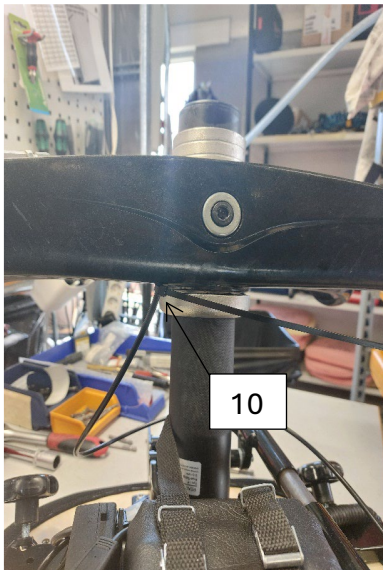

11. For additional information, refer to the following instruction: Leg half replacement instruction FSN 104121

Replacement of cable brakes FSN 104121. (revision 1)